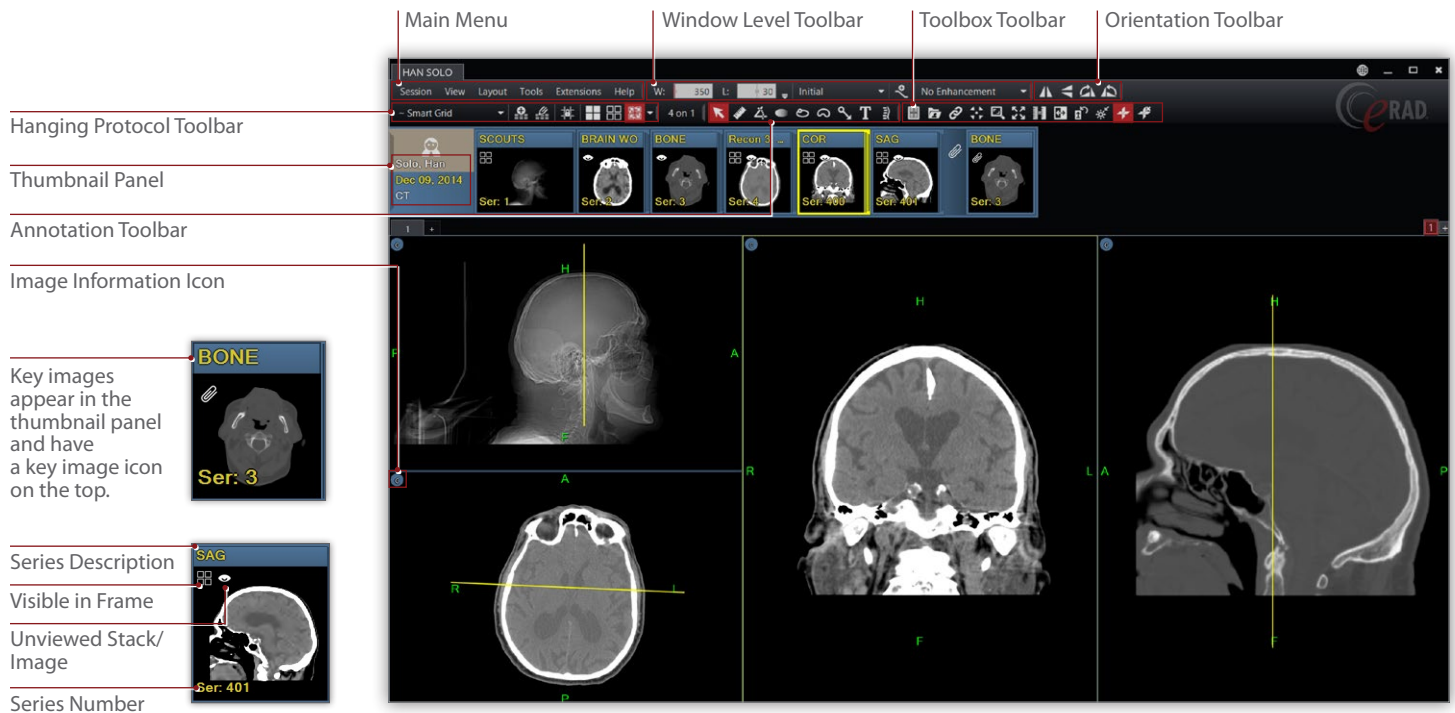


CONNECT

GET REPORTS | VIEW IMAGES | ORDER EXAMS | MAKE APPOINTMENTS

eRAD PACS Evo Image Viewer



Mouse Control

| | | |
|--|---------------------|---|
| | Scroll | Press Left click and hold on lateral border of image, move mouse up and down to scroll through image series. |
| | Window/Level | Press Right click and hold on image, move mouse up and down, or left and right to increase or decrease window level. |
| | Zoom | Press Right and Left click together and hold on image, move mouse up to zoom in and down to zoom out. Press F1 to see full Page View. |

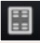









Annotation Toolbar

| | | |
|--|---------------------------|--|
| | Default Mode | General cursor mode, used to select an image or object within the display. |
| | Measure Distance | Perform linear measurements. |
| | Measure Angle | Perform angular measurements. |
| | Hounsfield Value | Calculate and display the average pixel value within the defined. |
| | Region of Interest | Calculate the area within the defined region of interest. |
| | Free Region | Calculate the area within the defined region of interest. |
| | Pointer | Add a point graphic and optional text string. |
| | Text | Define a text string and place it on the image. |
| | Spine Labeling | Display labels identifying vertebral disks and interspaces. |





Hanging Protocol Toolbar

| | | |
|--|----------------------------|---|
| | Smart Grid | Display applied hanging protocol. Expands hanging protocol selection list. |
| | Create Layout | Save the current layout as a hanging protocol template. |
| | Layout Edit Mode | Enable mode to customize layout frames. |
| | Lock Grid Lines | Enable or disable the ability to manually resize grid frames. |
| | Tile Matrix | Apply a tile pattern to a selected frame controlling the number of images displayed in the frame. |
| | Grids Matrix | Apply selected grid fill mode. |
| | Grid fill Mode List | Display selected grid fill mode and expands grid fill mode list. |
| | Empty Cells | Draw empty grid frames. |
| | Reused Cells | Draw grid frames and populate with currently displayed images. |
| | Selected Study | Draw grid frames and populate with selected study. |
| | All Study | Draw grid frames and populate with all studies loaded in viewer. |



Toolbox Toolbar

| | | |
|---|-------------------------------|---|
|  | Show Image Information | Show/hide the demographic and image overlay information. |
|  | Patient Folder | Access Patient Folder to View Reports and Prior Images for comparison. |
|  | Link Mode | Link one or more image frames together. |
|  | Magic X | Locate a defined point in one image on all other images that share the same frame of reference. |
|  | Magic Glass | Display a floating magnifying glass for zooming in on the image data behind it. |
|  | Fit to Window | Fit the selected image to the size of the frame. |
|  | Invert Grayscale | Invert the slope of the applied window level. |
|  | Reset Orientation | Reset the image(s) in the selected frame to the original orientation. |
|  | Reset Window/Level | Reset the image(s) in the selected frame. |
|  | Localizer Lines | The intersection of a referenced image on other views. Show All or Hide all localizer lines. |



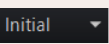

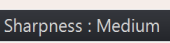
Orientation Toolbar

| | | |
|---|------------------------|---|
|  | Flip Horizontal | Flip image(s) in selected frame horizontally. |
|  | Flip Vertical | Flip image(s) in selected frame vertically. |
|  | Rotate Right | Rotate image(s) in selected frame to the right 90°. |
|  | Rotate Left | Rotate image(s) in selected frame to the left 90°. |

Session

| | | |
|---|--------------|------------------------|
|  | Save | Save Image or Series. |
|  | Print | Print Image or Series. |

Window/Level Toolbar

| | |
|--|--|
|  | Window width of the selected image frame. |
|  | Window level of the selected image frame. |
|  | Menu of preset window/level settings. |
|  | Save the current W/L settings as a modality specific preset. |
|  | The image enhancement list provides access to preset image sharpening, smoothing and edge detection. |

CT Preset Window/Level Hotkeys

| PRESET | TOP ROW KEYBOARD |
|----------|------------------|
| Initial | 0 |
| Soft | 1 |
| Liver | 2 |
| Lung | 3 |
| Brain | 4 |
| Subdural | 5 |
| Bone | 6 |
| Actual | 8 |
| Auto | 9 |