

Install the eRAD Evo Image Viewer Combo

- 1 Click the **Account** link on the main toolbar.
(gear icon near the top right).



- 2 Click the **Image Viewer** tab.

Account Settings


Image Viewer

- 3 Click the **Download Full Viewer** button.

DOWNLOAD FULL VIEWER


- 4 Go to your **downloads** folder and locate the **PACS-Viewer_Combo277.zip**

Right-click and **Extract All**.

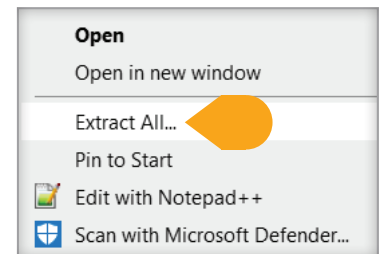

- 5 Select a destination.

Once completed open the
PACS-Viewer-Combo277 folder.

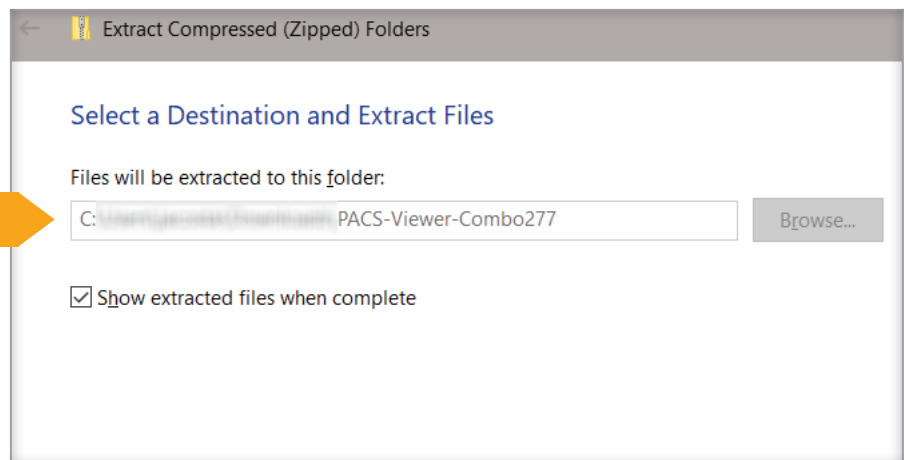

- 6 Launch the **PACS-Viewer-Combo.exe** file and follow the installation instructions.

After the installation you will be able to launch images.

If for any reason images do not launch on first try, please reboot the workstation.
You can now launch studies using the Evo viewer.

