










Please use this handy reference card to help you navigate the eRAD PACS Viewer.

If you have any questions or need an additional copy of this reference guide, please call 443.436.4230.


Tool Box Modes





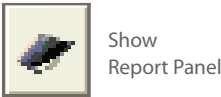



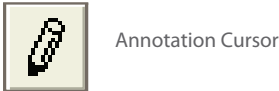

Normal Cursor			Image Magnify
Key Image Selector			Link Series
ROI with W/L			Cropping
Magic X			Use Magic Glass

Other Important Tools



Annotation Tools



Measure Distance			Measure Angle
Cobb Angle			ROI Hounsfield
ROI			Free ROI
Pointer			Text
Report Image Numbering			Spine Labeling

Layout Keys: Show / Hide*

Layout Manager	F6*	Image Overlay Info	F10*
Image Info Dump	F7*	Image Orientation	F11*
Report	F8*	Localizer Lines	F12*
Annotations	F9*	Thumbnail Panel	ALT + T

* All Function (F) keys can be toggled to show or hide by pressing them once or twice.
Their settings can be altered by use of Key Binding functionality located in settings menu.

Navigation Keys

Open All	Double-Click Thumb Name
Open All-Stack	Ctrl + Double-Click Thumb Name
Full Screen On / Off	F5*
Continuous Zoom	Press and Hold L & R Mouse Buttons
Image Pan When Zoomed	Press and Hold L Mouse Button
Fit to Window	Ctrl + F
Adjust Window / Level	Press and Hold R Mouse Button