


LAYOUT KEYS*

Layout Manager Show / Hide	F6
Image Info Dump Show / Hide	F7
Report Show / Hide	F8
Annotations Show / Hide	F9
Image Overlay Info Show / Hide	F10
Image Orientation Show / Hide	F11
Localizer Lines Show / Hide	F12
Thumb-Nail Panel Show / Hide	Alt - T


NAVIGATION KEYS

Open All	Dble Click Thumb Name
Open All-Stack	Ctrl Dble Click Thumb Name
Full Screen On / Off	F5*
Continuous Zoom	Press and Hold L & R Mouse Buttons
Image Pan When Zoomed	Press and Hold L Mouse Button
Fit to Window	Ctrl - F
Adjust Window/Level	Press and Hold R Mouse Button

TOOL BOX MODES

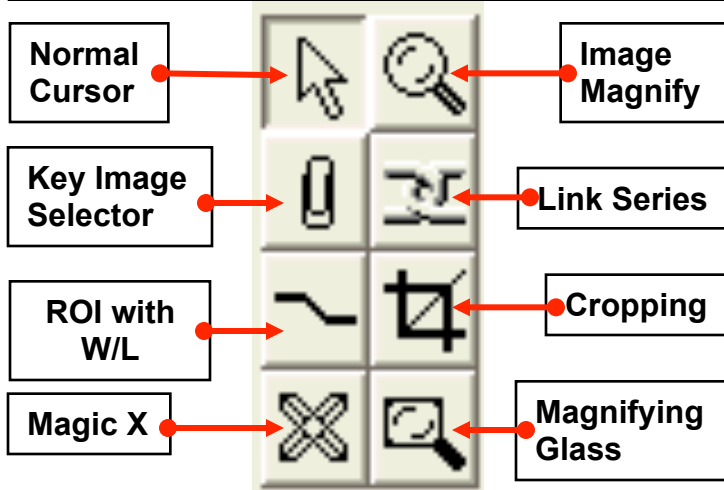

ANNOTATION TOOLS

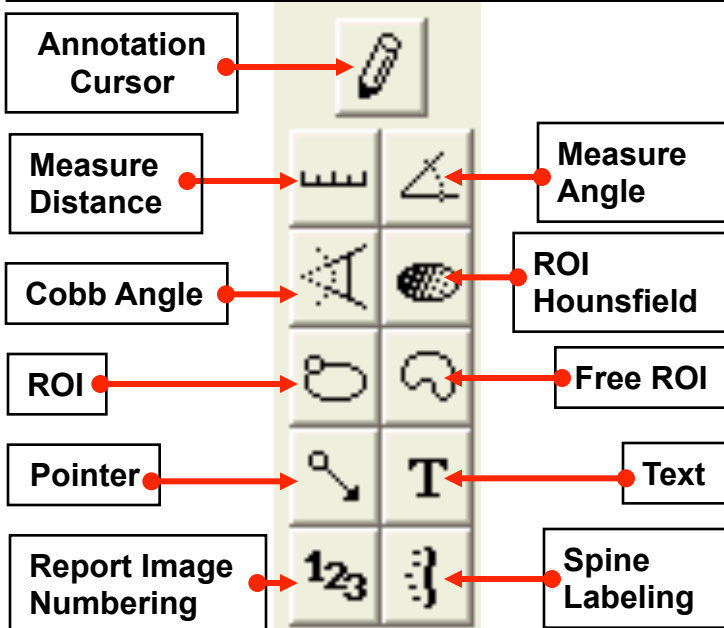

* All function key (F Keys) settings can be altered by use of **Key Binding** functionality located in settings menu